




INSTRUCTIONS FOR USE

Article 10 005 Apron



This instruction of use must be available to each user.

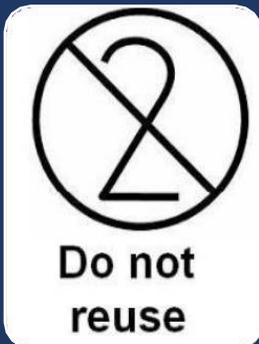

Model: Apron

Article: 10 005

Manufacturer: ACG Production AB

Country of manufacture: Sweden

CE Marking: CE PPE Category1 according to Regulation (EU) 2016/425 European Parliament and of the Council on protective equipment.

Plastic protective barrier: Protects against liquid splashes, contact with weak detergents, only to be used for protection against minimal risks.

Permeability: Impermeable to water and meets permeability requirements according to ISO 6529:2013, method A. <1 µg/cm²/min.

Usage: Disposable item.

Material: Non-Woven Spunbond Polypropylene LDPE (Low-density polyethylene) (70-75 g/rock).

EU Declaration: Available at www.acgproduction.se

Employers regulation: The employer must inform the employee in advance of the risks against which the personal protective equipment in question is to be protected.

Employers regulation: The employer must also arrange instruction and practice and, if necessary, demonstrate how the equipment should be used to achieve the intended protective effect. Appropriate written information for each personal protective equipment in question shall be provided and available within the undertaking or establishment.

Instructions for employees: The employee must follow given instruction when using the personal protective equipment.

Performance: Refers to standard ISO 13688:2013

EN 14605:2013 (applicable parts)

Storage: Store at room temperature, avoid extreme temperatures. Store in its unbroken packaging during transport

Recycling: Combustible waste

Instructions for use: Available at www.acgproduction.se

Shelf life: Unbroken packaging, 12 months from the date of delivery, Under the right stock conditions for the product, it can be useful for a long period of time.



TO PUT ON THE APRON

Take out the apron that is folded and roll it out so that the front is visible.

- Hold up the apron. Thread the apron over your head.
- Pull off the belts..



Pull and adjust any creases on the abdomen.

- Tie the apron firmly in the back.
- The apron should fit securely.



TO REMOVE THE APRON

Take a firm grip on the fabric at the neck. Pull with a powerful jerk downwards and outwards.

- Grab the front of the apron at the waist.
- Pull with a strong jerk for the belt in the back to drop.



Pull the apron away from you.

- Discard the apron.
- Do not reuse the apron.



ACG Production AB
Älvsborgsleden 7
501 10 Borås
Sweden

E-mail: info@acgproduction.se
Website: www.acgproduction.se

